# FARMAC-ZABBAN S.p.A.

Via Persicetana, 26 – Calderara di Reno (BO)  
Tel. +39 051-318411 - Fax +39 051 – 318472  
e-mail : [company@farmaczabban.it](mailto:company@farmaczabban.it)

ST108-02\_IFU

Rev. 00  
01/09/21

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STERILE EO

Rev 00 del 01/09/21

**EN Luer Lock Safety Syringe with Exchangeable Needle**

**Instructions for Use**

1. Peel package as indicated to allow proper removal of SICURTOUCH Luer Lock Safety Syringe with exchangeable needle and remove needle cap.
2. Prepare vial for medication draw up.
3. Penetrate center of vial with needle and draw up desired dose of medication. (See Figure 1).
4. Prepare injection site and change needle if desired.
5. Administer medication per facility protocol.
6. To engage one hand safety feature activation and deliver the full dose, push the plunger through the zero line. (See Figure 2)
7. Pull back on the plunger until the locking ring is engaged. Observe withdrawal of needle into the barrel. You can pull back and lock the plunger with one hand only. (See Figure 3)
8. Snap off plunger to permanently disable syringe, you can snap off the plunger with one hand only. (See Figure 4)

**Single-handed safety mechanism activation with irreversible locking mechanism, confirmed with an audible click.**

**Caution:**

- \* This device is sterile unless the package has been damaged or opened.
- \* If unable to activate safety mechanism, discard syringe immediately into an approved sharps container.
- \* This device is single use only.
- \* Maximum needle length cannot exceed 1 1/2" or 38.1 mm.

Any serious accident occurring in relation to the medical device must be reported to the manufacturer and the competent authority of the Member State in which it is based. Store in a cool and dry place, away from heat or open flames and inside the original packaging. Avoid dust environments and do not expose to direct sunlight. Federal (USA) law restricts this device to sale by or on the order of a physician or other licensed practitioner.

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STERILE EO

Rev 00 del 01/09/21

**EN Safety Syringe with fixed needle**

**Instructions for Use**

1. Peel package as indicated to allow proper removal of SICURTOUCH Safety Syringe with fixed needle and remove needle cap.
2. Prepare vial for medication draw up.
3. Penetrate center of vial with needle and draw up the desired dose of medication. (See Figure 1).
4. Prepare injection site and administer medication per facility protocol.
5. To engage the safety feature and deliver the full dose push the plunger through the zero line. (See Figure 2).
6. Pull back on the plunger until the locking ring is engaged. Observe withdrawal of needle into the barrel. You can pull back and lock the plunger with one hand only. (See Figure 3)
7. Snap off plunger to permanently disable syringe, you can snap off the plunger with one hand only. (See Figure 4)

**One-handed safety system with irreversible audible and tactile locking mechanism**

**Caution:**

- \* This device is sterile unless the package has been damaged or opened.
- \* If unable to activate safety mechanism, discard syringe immediately into an approved sharps container.
- \* This device is single use only.
- \* Maximum needle length cannot exceed 1 1/2" or 38.1 mm.

Any serious accident occurring in relation to the medical device must be reported to the manufacturer and the competent authority of the Member State in which it is based. Store in a cool and dry place, away from heat or open flames and inside the original packaging. Avoid dust environments and do not expose to direct sunlight. Federal (USA) law restricts this device to sale by or on the order of a physician or other licensed practitioner.

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